



ThanoSeal Frequently Asked Questions Version 5.0

What does ThanoSeal™ do?

ThanoSeal™ is an occlusive sealant for incisions, abrasions, lacerations and skin slip. ThanoSeal™ will halt fluids from leaking through or around any wound or cut and completely seals the site.

How do I apply ThanoSeal™?

Dry the skin in the application area, select either a patch or tape size that covers the area. Be sure to include enough surrounding skin to seal the entire area.

Can ThanoSeal™ be applied to wet tissue?

No, it prevents the adhesive from sealing the wound, cut or abrasion.

Where can I purchase ThanoSeal™?

You can purchase ThanoSeal™ from Argent Distributors.

Crescent Sales in Mississippi by calling: **(888) 856-4516**

Southland Medical in So. California by calling: **(800) 959-9160**

Can I handle ThanoSeal™ directly with my gloves?

Yes, however, the less contact, with objects other than the target application area, the better.

Does the application area for ThanoSeal™ need to be shaved?

No, it does not in most cases. If the area has particularly thick hair it would be advisable to remove it to maximize contact with skin for optimal sealing effect.

We use low temperature embalming protocols, can ThanoSeal™ still be applied?

Yes. ThanoSeal™ is rated to be applied to tissue at 55 degrees Fahrenheit or higher. If ThanoSeal™ is applied above the rated temperature and sealed; it can be placed into a low temperature environment without losing its seal. If the case is embalmed and placed into low temperature environment (below 55 degrees F) without applying ThanoSeal™ then the case needs to be removed from low temps and allowed to warm to rated temperature. *Most state requirements for refrigeration are 38 degrees to 40 degrees F. given the average time of embalming, the surface temperature of the tissue will generally reach application temperatures or above.

ThanoSeal Frequently Asked Questions Version 5.0 (Continued)

How can I get a discount on ThanoSeal™ tape?

Call your distributor and discuss pricing with them. Many funeral homes have specially priced contracts with the distributors.

Is ThanoSeal™ tape expensive?

ThanoSeal is priced 60-70% below comparable products for the medical industry. ThanoSeal™ tape was developed for the funeral home industry and when you use ThanoSeal you will discover that you begin using less of other products for sealing these wounds. In addition, the use of ThanoSeal will save you a great deal of time. It will also provide “insurance” that the embarrassing situation of leaking will be greatly reduced, allowing you to provide a better experience to the family. Call backs for correcting leakage are reduced, saving even more time.

Does ThanoSeal™ tape come in different colors?

Not today, we are developing various skin tones for future release as well as a facial tape.

Can ThanoSeal™ tape be used with make-up?

Yes, make-up can be applied directly to ThanoSeal tape.

Does ThanoSeal™ have an odor?

Yes, a very pleasant “vanilla” scent.

Can I apply ThanoSeal™ tape on living skin?

It is not recommended for use on living skin.

Can I return unused product?

Yes, unused product can be returned if it is unopened, clean, and dry with a 15% restocking charge if acquired from Argent International directly. If acquired from your distributor, their terms for return apply.

Does ThanoSeal™ have shelf life?

ThanoSeal™ has a shelf life of 1 year.

Does storage temperature impact the effectiveness of ThanoSeal™?

ThanoSeal™ should be stored in a cool dry place below 76° F (24°C). Rolls should be stored on end.

Does sunlight effect ThanoSeal™?

ThanoSeal™ is not an ultraviolet stable product. You must keep it out of the sunlight. ThanoSeal is for indoor use only.